

Proven protection from
Sudden Cardiac Death
+ so much more



The LifeVest Network Patient Data Management System

The LifeVest provides proven protection from Sudden Cardiac Death with a connection to your patient's changing cardiac condition through the LifeVest Network.

The LifeVest Network is ZOLL's online patient data management system that allows clinicians to monitor patient data downloaded from a patient's LifeVest wearable defibrillator. The LifeVest Network enables you to have access to the most current information about your patient during their recovery after a cardiac event.

Data captured by the patient's LifeVest device is transmitted to the secure LifeVest Network website where it is archived for review. Information on the LifeVest Network can be accessed anytime from almost anywhere.

The LifeVest Network

Customizable

The LifeVest network allows you to monitor your patients efficiently and has been designed to fit seamlessly into your practice. You can customize the way you view and are notified of patient events recorded by the patient's LifeVest device.

Flexible and Fast


Utilizing powerful and flexible software technology, the LifeVest Network allows you to easily and quickly obtain LifeVest patient data. Prescribers of the LifeVest also have the option of assigning access to other clinicians in their practice to take a more active role in reviewing LifeVest patient data.

Patient Management Features

Efficiently Triage LifeVest Patients

The LifeVest Network allows you to customize the way you view and are notified of patient events recorded by the patient's LifeVest device.


The Dashboard allows for efficient evaluation of patient data.

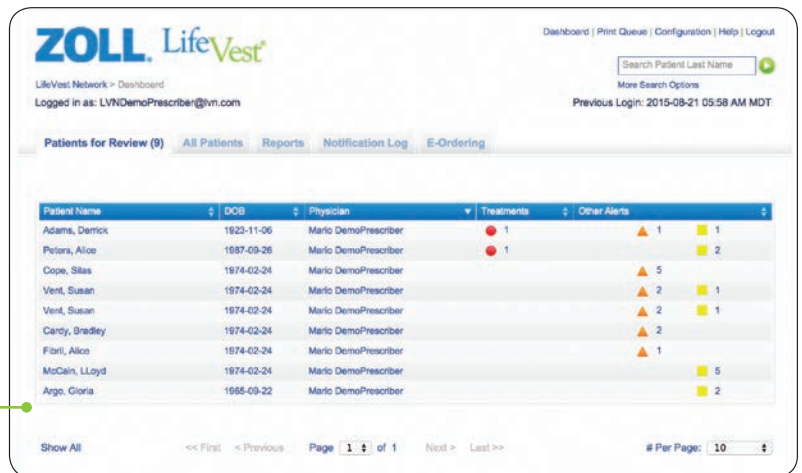

Tailor Alerts

The LifeVest Network gives you additional options to determine what types of LifeVest patient information necessitate an alert. You can choose to be notified for events such as:

- Treatments
- Patient Recorded ECG
- Detected arrhythmia where no treatment was given (e.g. short run of VT)

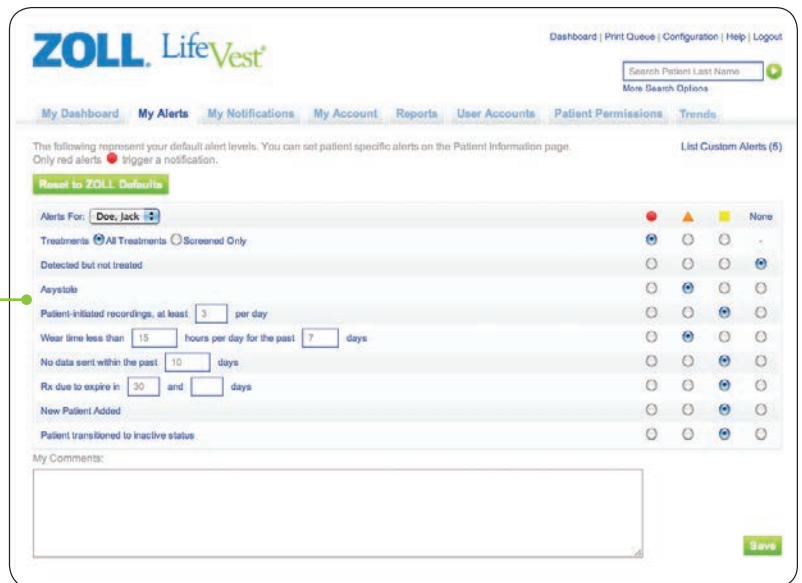

Set Notifications


Physicians and clinicians can also decide how they, or their delegate, wish to receive notifications whether by phone, fax, SMS, and/or email, as well as the times of day they would like to receive these notifications (e.g. different settings for office hours and after-hours).

ECG Review

ECG recordings automatically recorded by the LifeVest or manually captured by the patient provide additional information that can be used to decide future care paths.

ECG information captured by the LifeVest can help you diagnose:

- Sustained VT/VF
- Non-sustained VT
- Atrial arrhythmias/SVT
- Severe bradycardia/Asystole

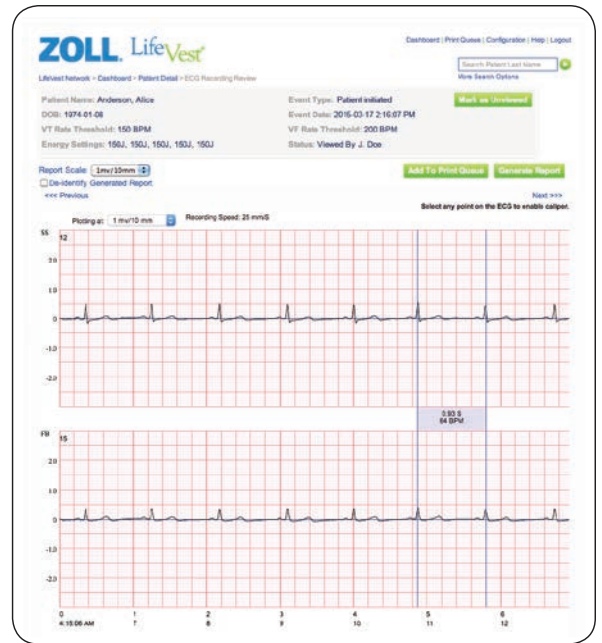

Trends

LifeVest Network Trends provides you with continuous insight into your patients' health status and response to therapy during the early period after a cardiac event, offering information that you can use to optimize their care.

Information about your patient are trended over time, indicating when a change in their status may require your action.



Heart Rate

Heart Rate displayed as weekly, daily, or five minute averages

Activity Measures

- Patient Activity (steps taken)
- Body position
- Body angle while at rest

Health Survey

The optional Health Survey allows you to select up to 12 questions that your patients can answer through the LifeVest's touchscreen display.

e-Ordering

You can efficiently order the LifeVest, or renew an existing medical order, through the LifeVest Network's e-Ordering interface. Once submitted, the LifeVest medical order is transmitted directly to ZOLL for processing.

The e-Ordering functionality allows for coordination of efforts of the clinical staff (e.g., upon direction from a physician, a staff member may enter information into the online medical order form, attach proper documentation, and forward to the prescriber for review and electronic signature.)

Dashboard | Print Queue | Configuration | Help | Logout

LifeVest Network > Dashboard > E-Order

Logged In as: LVNDemoPrescriber@lvn.com

Previous Login: 2015-08-21 12:17 PM EDT

Search Patient Last Name

More Search Options

* Prescribing Physician: DemoPrescriber, Beth * NPI: Lookup NPI

* Patient Name: First Name MI Last Name * Date of Birth: Year Month Day

Primary Insurance: Secondary:

* Reason for LifeVest: Select Primary Reason Select ICD-9 Code; if "other," enter code below

Upload Supporting Documentation: Choose File No file chosen Attach File

Attached Supporting Documentation

* Estimated Length of Need: * Estimated Start Date:

* VT Rate Threshold, BPM: 150 Default * VP Rate Threshold, BPM: 200 Default Reset to Defaults

* Energy, Joules: Shock 1 Shock 2 Shock 3 Shock 4 Shock 5

Comments: Please include in comments where the patient is to be fit (e.g. Heart Care Hospital, room 320).

Prescribing Physician Electronic Signature

Prescriber Login: Prescriber Password:

I, Beth DemoPrescriber, hereby acknowledge that by entering my LifeVest Network password and clicking "Submit Order", 1) I am placing a binding order for LifeVest® Wearable Defibrillator on the LifeVest Network Electronic Ordering Terms and Conditions of Use set forth below; 2) I am confirming the associated order information provided above is accurate and complete to the best of my knowledge and 3) I am certifying that I have read and accepted the LifeVest Network Electronic Ordering Terms and Conditions of Use.

I have read and understand the statement above.

E-Order LifeVest OR **Fax Order LifeVest**

- Electronically submit medical order
- ZOLL will begin processing

- Generates PDF of medical order with fax cover page
- PDF must be printed, signed by prescriber, and faxed to ZOLL.

Save and Resume Later

* Denotes Required Field

Contact **ZOLL** 24 hours a day, 7 days a week



For LifeVest customer support, technical support or medical orders, please call **800.543.3267**



LifeVest medical orders and supporting documentation can be faxed to **866.567.7615**



For more LifeVest information, please visit **www.zoll.com** or email **LifeVest.Info@zoll.com**



Access LifeVest patient data and e-order through **LifeVestNetwork.Zoll.com**

ZOLL LifeVest®